


Proper care instructions for your new treatment head

Thank you for your purchase! Please let us acquaint you with some of the new features of our treatment head.

The bulbs used in our treatment heads are made from lead free glass, and annealed to prolong its lifespan. Inside the bulb we use a proprietary formula of Krypton, Argon, and Xenon that has been proven to be most effective. The bulbs have been designed to be removable, in the event of accidental damage. The bulb is attached to the case with a thick rubber gasket to prevent stress on the glass and to resist damage when dropped. We use a strain relief to insure the connection from the cord to the case for longevity. We include a 10 foot long cord for easier mobility. We include a protective cover to insure a long life.

Caring for your treatment head!

Use 70% rubbing alcohol and a cotton ball or gauze to clean the exterior in between treatments. Do not douse or immerse the bulbs, and allow the bulb to completely dry before the next treatment. Do not place the treatment head near any electronics. Remove the protective covering before each use. Do not kink the cable nor bend it.

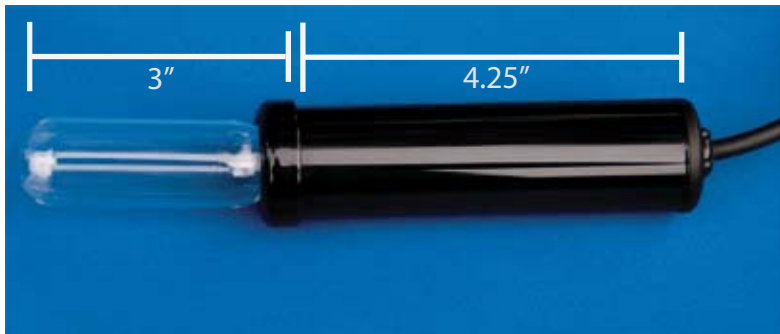

Best used with our exclusively designed equipment




Shown with protective cover


LYMPH DRAINAGE XP™



LYMPH DRAINAGE XP2™

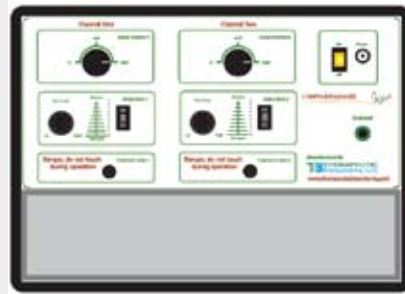


LYMPH DRAINAGE HOME EDITION™

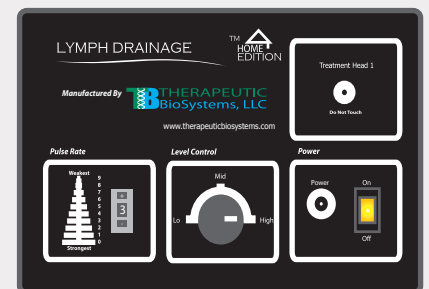

Treatment Heads

Replaceable Bulbs
Drop Resistant
Flexible Casing
10' Cord
Annealed Glass

Lead-Free Glass Bulbs

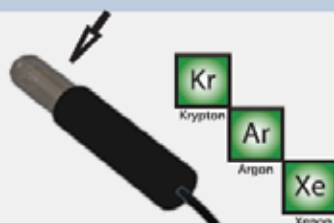

Distributed By: